
---

## EasyViewer Full HD

## Quartz Full HD

*Model: Vocatex 3*

---

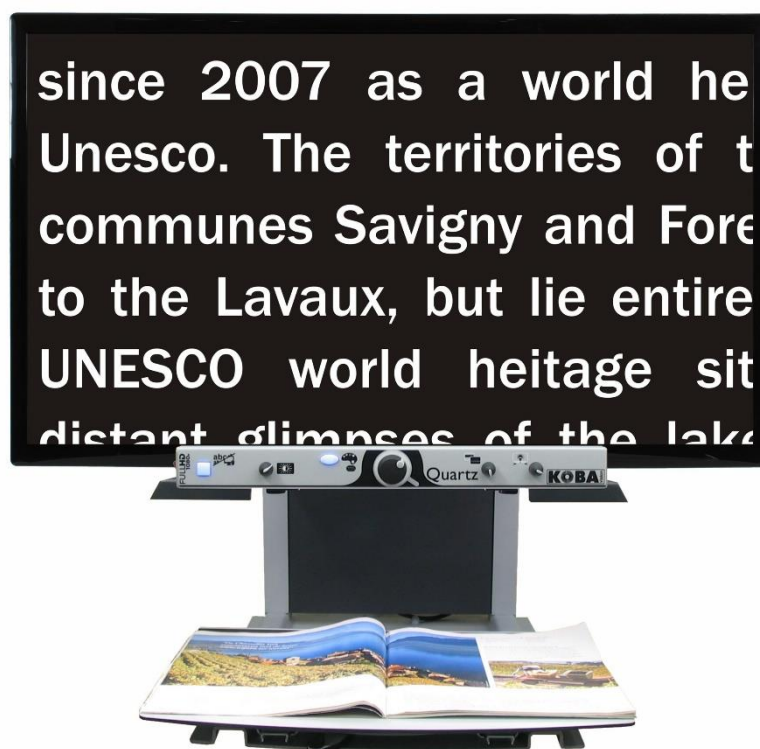**Français - Mode d'emploi - Page 1**  
**Nederlands – Gebruikshandleiding - Pagina 10**  
**English - User Manual - Page 19**  
**Deutsch – Gebrauchsanweisung - Seite 28**  
**Specifications 37**

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# English – User Manual

*This manual describes the KOBA Vision EasyViewer and KOBA Vision Quartz: EasyViewer. The latest versions of the manual can be downloaded from the KOBA Vision website: <http://www.kobavision.be>.*

*Release date: September 2017.*

## **Chapter 1: Safety and maintenance instructions**

### **SAFETY INSTRUCTIONS**

- The device is not to be used in a medical environment.
- Never sit on the sliding table.
- Never attempt to pull yourself up by holding the EasyViewer/Quartz.
- The power supply must only be replaced by the same type (15V, max. 2,5A, EN60601-1) after switching off the device and removing the adapter from the grid.
- The connections have been designed to only connect in 1 way. Never force a connection. The cables are only to be connected by authorised persons. Install the power supply of the device in a well ventilated area.
- Never attempt to repair the device by yourself. It is dangerous to remove protective covers which are there to prevent electric shock. You must contact KOBA Vision or your dealer in case of defects.
- To avoid electrocution, do not place the device in a spot where it can come into contact with liquids.
- When moving the device, the height adjustment must always be blocked by using the supplied orange screws.
- Consult the user manual of the screen for specific safety instructions related to the screen.
- Save this manual carefully. It contains all necessary information about your device.
- Keep the device in a dry, well-ventilated and clean place. Do not expose the device to direct sunlight or a heat source.
- Never use this device in ambient temperatures higher than 35°C.
- This device is designed as aid to magnify and to read aloud documents. Any change of purpose voids the CE-marking.
- Your product is designed and manufactured with high quality materials and components, which can be recycled and reused. Do not dispose of your old product in your general household waste bin. Dispose of the complete product (including its cables, plugs and accessories) in the designated collection facilities.


# WARNING

The screen is to be assembled and disassembled by authorized persons only. Incorrect installation or assembly can cause severe damage and/or injuries.

You must be extremely cautious when moving the EasyViewer/Quartz. The EasyViewer/Quartz must not be picked up by the screen, the front panel, the light casings or the slide table. Always block the height adjustment with the supplied orange screws.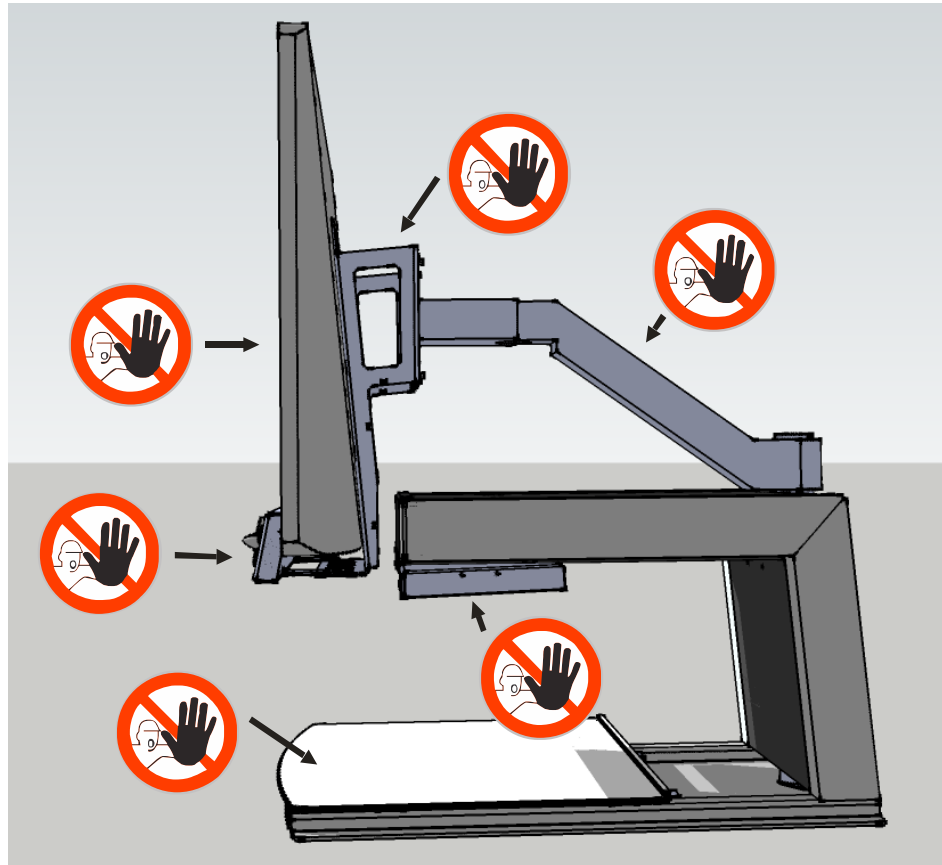

## MAINTENANCE INSTRUCTIONS


- Occasionally clean the screen and the lens using a damp cloth. For further instructions: check the screen manual.
- Treat your device with care and you will enjoy it for years to come.

## *Chapter 2 : Installation*

The EasyViewer/Quartz is solely to be installed by authorised personnel. You can find a list on [www.kobavision.be](http://www.kobavision.be)

# Chapter 3: Controls

## OPERATING PANEL

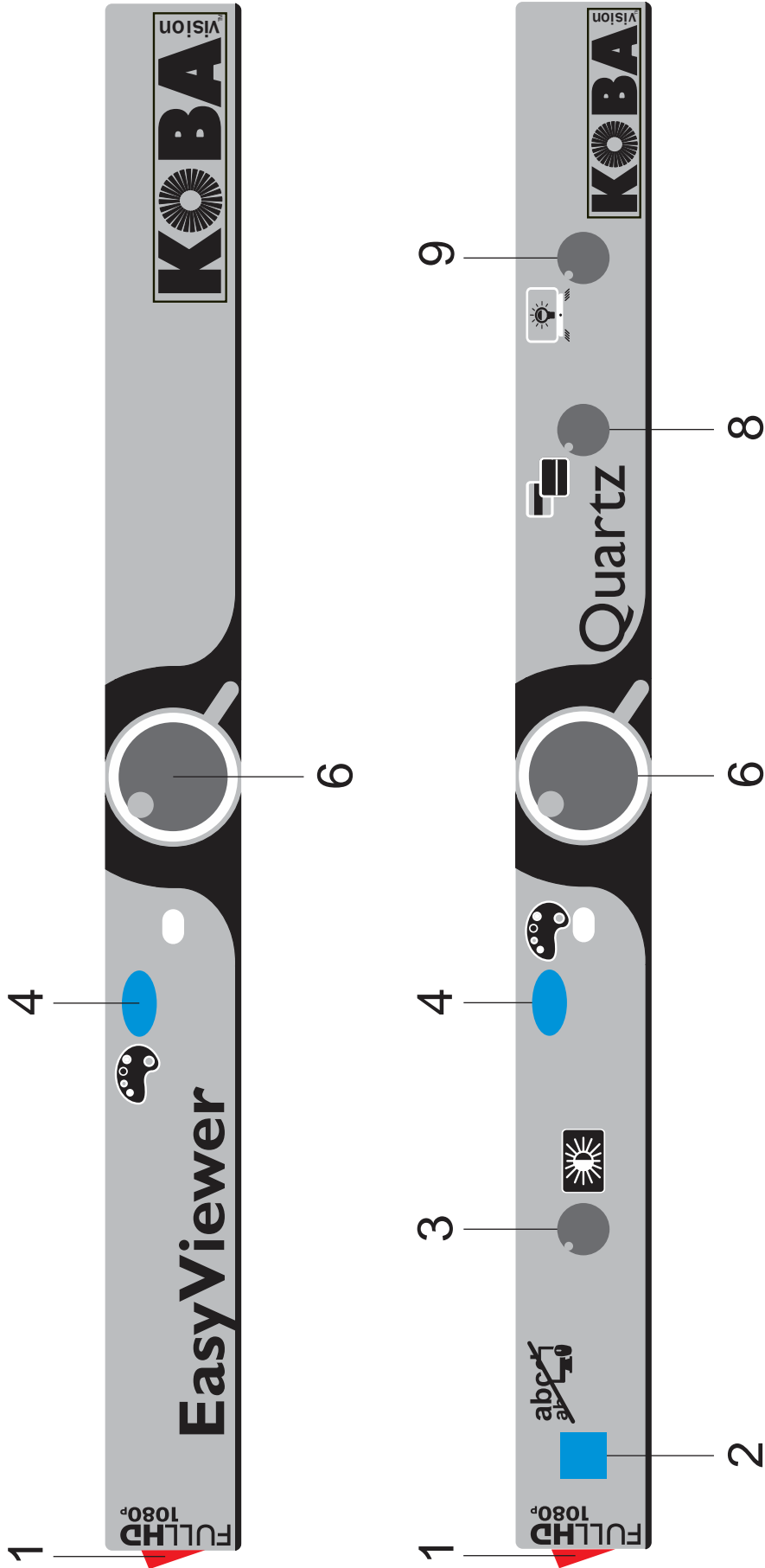1	ON/OFF switch
2	Switch computer/Quartz
3	<b>Press:</b> on/off spoken buttons <b>Turn in colour mode:</b> brightness <b>Turn in positive or negative reading :</b> improve background
4	<b>Press briefly:</b> switch between photo mode, negative, positive reading <b>Hold button:</b> changes colour of the background or characters
6	<b>Turn:</b> zoom in/zoom out <b>Press:</b> overview/magnification
8	<b>Turn:</b> set the position of the line or bar <b>Press:</b> switch between line, bar or none <b>Press and turn:</b> adapt the line colour or adapt bar width
9	<b>Turn:</b> adjust lighting

## SWITCHING ON/OFF

The device is switched on with **button 1**. Most screens are switched on by the EasyViewer/Quartz. If this is not the case, switch on the screen as well (see screen manual).

The EasyViewer/Quartz is switched off with **button 1**. Some screens switch off the EasyViewer/Quartz when they are switched off.

After switching on, an on-screen bar shows the start-up progress. The EasyViewer/Quartz indicates when it is ready to be used by telling “the device is ready for use”.

## LANGUAGE OF THE SPOKEN HELP MESSAGES

Attention! This menu will pop-up when initializing the unit for the first time.

### Step 1: Language of the spoken assisting messages

This is where you adjust the language of the spoken buttons. You choose your mother tongue. An inhabitant of France chooses French. By turning the **button 6** you choose the language. Hold **button 4** to go to step 2.

### Step 2: Preferred voice

Most languages have several voices. In step 2 you choose your preferred voice for the spoken buttons and your preferred voice for reading out the texts by turning **button 6**. Usually you can choose between a male and female voice. In French, for example, you can choose between Virginie, Thomas, Felix and Audrey. You select the language by turning the big knob. By holding **button 4** you will exit the menu.

# COMPUTER CONNECTION

**Button 2** of the EasyViewer/Quartz switches between HDMI 1 and HDMI 2, or HDMI 1 and HDMI 3.

To use the button on the front panel, the EasyViewer/Quartz needs to be connected on HDMI 1, your computer needs to be connected on HDMI 2 or 3 of the screen. Connect with a HDMI-HDMI, HDMI-DVI or HDMI-DisplayPort cable.

The computer connection of the EasyViewer/Quartz has 2 possibilities:

## Full screen (most common)

By pressing the pc button on the EasyViewer/Quartz, the screen switches to your computer. The full computer image is displayed on your screen. Recommended resolution: 1360x768 or 1280x720.

## Split Screen

*(in combination with ZoomText 10 and higher or SuperNova 15.03 and higher) – only Windows OS*

With the split screen possibility you can use a part of the screen to display the EasyViewer/Quartz image and another part to display your computer. The video image can simply be put on top, bottom, left or right while having you computer image on the same screen. When working in split screen mode you will have full access to your computer.

The video image is send with a USB second cable to your computer. Use the **button 2** to switch to full screen. Open ZoomText or SuperNova and start the camera function. The video image is displayed.

### Specifications Computer

- Unused USB port (USB 3.0 or 2.0)
- Windows 10, 8.1, 8, 7 – 32/64 bit;
- ZoomText 10 and higher or SuperNova 15.03 and higher.

### Installation

- Connect the Quartz to the computer via the USB cable (USB 3.0 preferred);
- Configure the Quartz as camera under ZoomText or SuperNova;
- Start ZoomText or SuperNova webcam.

# PROGRAMMABLE BUTTONS

The EasyViewer/Quartz has programmable buttons. **Buttons 2, 4** and **6** can have different functions and possibilities. These functions can be activated by your dealer.

### **Button 2:**

- Switch between magnifier and computer (default)
- *Reset*

### **Button 4:** Different colour combinations

- Black/White, White/Black, Black/Yellow, Yellow/Black, Black/Green, Green/Black (default)
- *Black/Red, Red/Black, Black/Blue, Blure/Black, Black/Cyan, Cyan/Black, Black/Magenta, Magenta/Black, Black/Orange, Orange/Black, Black/Gray, Gray/Black, Blue/Yellow, Yellow/Blue, Blue/White, White/Blue, Red/White, White/Red, Orange/White, White/Orange, Green/White, White/Green*

### Button 6:

- Overview/magnification (default)
- *Switch between magnifier and computer, Reset, Reading aloud*

## LIGHTING

By turning **button 9** you can set the brightness of the camera and button lights.

## READING LINE

The horizontal reading line is a line from left to right on the screen.

Press **button 8** until the line is displayed on the screen. Turn **button 8** to set the position. Press and turn **button 8** to change the colour.

## READING BAR


The horizontal reading bar is a clear bar from left to right on the screen. The text that isn't covered by the bar is slightly visible.

Press **button 8** until the bar is displayed on the screen. Turn **button 8** to set the position. Press and turn **button 8** to change the width.

## HEADPHONE

A headphone or external speakers can be connected at the EasyViewer/Quartz (see mini-jack connector on the back).

## SLIDING TABLE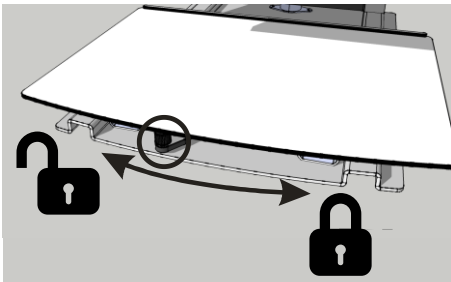


Under the table you can find a lever which controls the brake. If you move the lever to the left, the slide table is free to move left/right/up/down. If you move the lever to the right you adjust the friction. If you move the lever all the way to the right, the slide table will be blocked.

## HEIGHT AND TILT ADJUSTMENT


### Height (Flex arm and large screens)

According to personal preference, the screen of the Vocatex can be moved up or down. To do this, make sure you take hold of the screen on both the left and right side of the screen.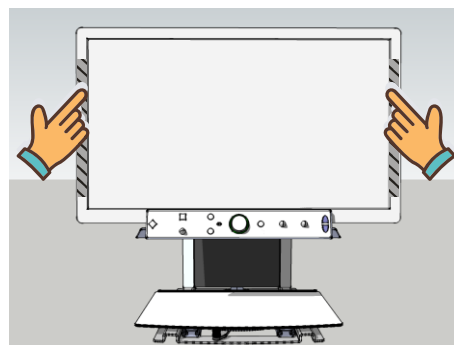

### Increase Lift Strength (Flex arm)

If the screen doesn't stay up when raised, increase the lift strength by turning the angle key right.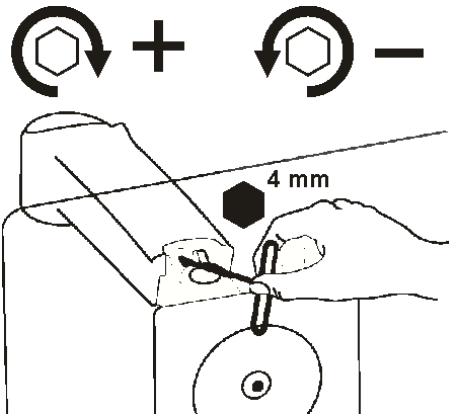

### Decrease Lift Strength (Flex arm)

If the screen doesn't stay down when lowered, decrease the lift strength by turning the angle key left.

## Tilt (Flex arm)


According to personal preference, the screen of the EasyViewer/Quartz can be tilt. To do this, make sure you take hold of the screen on both the left and right side of the screen.

### Increase Friction

If the screen tilts down, then you'll need to increase friction by turning the angle key right.

### Decrease Friction

If the screen is too difficult to tilt, then you'll need to decrease friction by turning the angle key left.

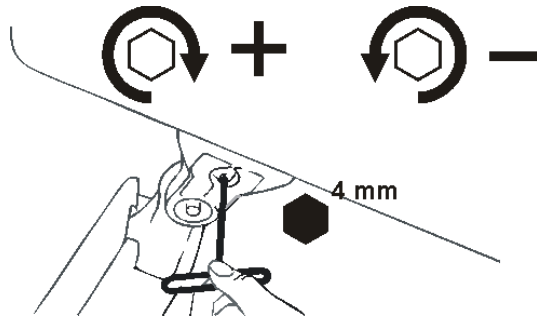## USAGE TIPS

### Reading

Reading is best done in positive or negative mode. These modes adapt the image for the visually impaired to a high-contrast image with improved background. Background and letters can be set at your discretion. The background can be set by means of background suppression (Quartz). This will allow you to filter out the structure of a sheet. This unique KOBA Vision function allows you at all times to set a high contrast background without annoying paper impurities. A newspaper can be read without annoying paper impurities. Factory settings will take you back to the original settings.

### Watch photos

Photos are best watched in photo mode. This mode is a unprocessed image without contrast enhancement. By changing the brightness (Quartz), you can adjust the image so that the photo becomes clearly visible.

### Crafting

Crafting is best done with a small magnification. It is better to select a dark background during this activity so that the camera is not hindered by large differences in contrast with whatever is darkened by the hands.

## Writing

You can also write under your video magnifier. It is best to use a black pen and lined paper.

# Chapter 4: Troubleshooting

Below is a list of problems that may occur with your EasyViewer/Quartz. Consult the list first before contacting KOBA Vision or your dealer.

Symptom	Symptom	Symptom
The device isn't turning on	No display on the screen	No sound
Possible causes	Possible causes	Possible causes
No power	Power failure The screen is not switched on	The screen volume is on zero The sound has been disrupted The headphone is plugged in The HDMI connector is loose
What to do	What to do	What to do
Check the power cable Check the adapter and verify if a green led is burning	Check whether the plug of the power cord has been properly plugged into the wall socket. If the device still won't work, pull the plug out of the wall socket. Then wait 60 seconds before plugging it back into the wall socket. Turn the screen on again. Press the "on" button on the remote	Slightly turn up the screen volume Check whether the sound has not been disrupted. Point the remote control directly towards the sensor at the front of the screen. Unplug the headphone Check the HDMI plug.

# Chapter 5: Warranty

## WARRANTY

KOBA Vision offers a two-year warranty on every KOBA VISION video magnifier, starting from the delivery date. To be able to benefit from the warranty, you need to keep the box and the packaging material.

Warranty services are provided by KOBA Vision or dealers acting on behalf of KOBA Vision. When the video magnifier cannot be repaired correctly, KOBA Vision reserves the right to replace the reading device by an equal device. All components and products replaced within the warranty period become property of KOBA Vision.

## WARRANTY PROVISIONS

The warranty becomes invalid if the video magnifier:

- Was not used according to instructions or was misused;
- Was damaged as a result of a fall or impact;
- Was disassembled or tinkered with;
- Was repaired earlier by a non-official service partner.

Consequential loss is under no circumstances covered by the warranty.

KOBA Vision will neither assume any liability whatsoever nor accept any possibility of liability except for the warranty provisions explicitly stated above. This limitation to the liability does not imply a breach of the customer's legal consumer rights or his or her rights vis-à-vis the vendor.

## **SERVICE FULFILMENT**

Before claiming the warranty or service and before contacting KOBA Vision or a dealer, you must note product name, serial number, purchase date, invoice number and any error messages or particular problem aspects.

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