Vocatex Standard Full HD

Vocatex Plus Full HD

Model: Vocatex 3

famous as a
on on the shores
man) in western
territory roughly
ux, extending to
0 meters above
g 6 lakeshore
evey. Part of the

since 2007 as a world heritage site by Unesco. The territories of two additional communes Savigny and Forel, also belong to the Lavaux, but lie entirely outside the UNESCO world heritage site. With only distant glimpses of the lake, at 700-900 meters altitude, along with the upper reaches of the commune of Puidoux, these

above the Léman
wine-growing, but
benefit from stunning
entire mountain range
Moléson of the Fribourgeois
to the Dent D'Oche
Français. In winter
is often sufficient
that it is not unusual
off-piste cross-country
rolling farmland
forest trails. In Autumn
Forel may bask in sun
on the days when La

ince 2007 as a world heritage site by Unesco. The

Français - Mode d'emploi - Page 1
Nederlands – Gebruikshandleiding - Pagina 13
English - User Manual - Page 25
Deutsch – Gebrauchsanweisung - Seite 37
Specifications 49

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English – User Manual

This manual describes the following versions of the KOBA Vision Vocatex: Vocatex Standard, Vocatex Plus. The latest versions of the manual can be downloaded from the KOBA Vision website: www.kobavision.be.

Release date: November 2018.

Chapter 1: Safety and maintenance instructions

SAFETY INSTRUCTIONS

- Text read by the Vocatex is under no circumstance guaranteed to be a 100% correct. The text-to-speech function is solely to be used as a reading aid.
- The device is not to be used in a medical environment.
- Never sit on the sliding table.
- Never attempt to pull yourself up by holding the Vocatex.
- The power supply must only be replaced by the same type (15V, max. 2,5A, EN60601-1) after switching off the device and removing the adapter from the grid.
- The connections have been designed to only connect in 1 way. Never force a connection. The cables are only to be connected by authorised persons. Install the power supply of the device in a well ventilated area.
- Never attempt to repair the device by yourself. It is dangerous to remove protective covers which are there to prevent electric shock. You must contact KOBA Vision or your dealer in case of defects.
- To avoid electrocution, do not place the device in a spot where it can come into contact with liquids.
- When moving the device, the height adjustment must always be blocked by using the supplied orange screws.
- Consult the user manual of the screen for specific safety instructions related to the screen.
- Save this manual carefully. It contains all necessary information about your device.
- Keep the device in a dry, well-ventilated and clean place. Do not expose the device to direct sunlight or a heat source.
- Never use this device in ambient temperatures higher than 35°C.
- This device is designed as aid to magnify and to read aloud documents. Any change of purpose voids the CE-marking.
- Your product is designed and manufactured with high quality materials and components, which can be recycled and reused. Do not dispose of your old product in your general household waste bin. Dispose of the complete product (including its cables, plugs and accessories) in the designated collection facilities.


WARNING

The screen is to be assembled and disassembled by authorized persons only. Incorrect installation or assembly can cause severe damage and/or injuries.

You must be extremely cautious when moving the Vocatex. The Vocatex must not be picked up by the screen, the front panel, the light casings or the slide table. Always block the height adjustment with the supplied orange screws.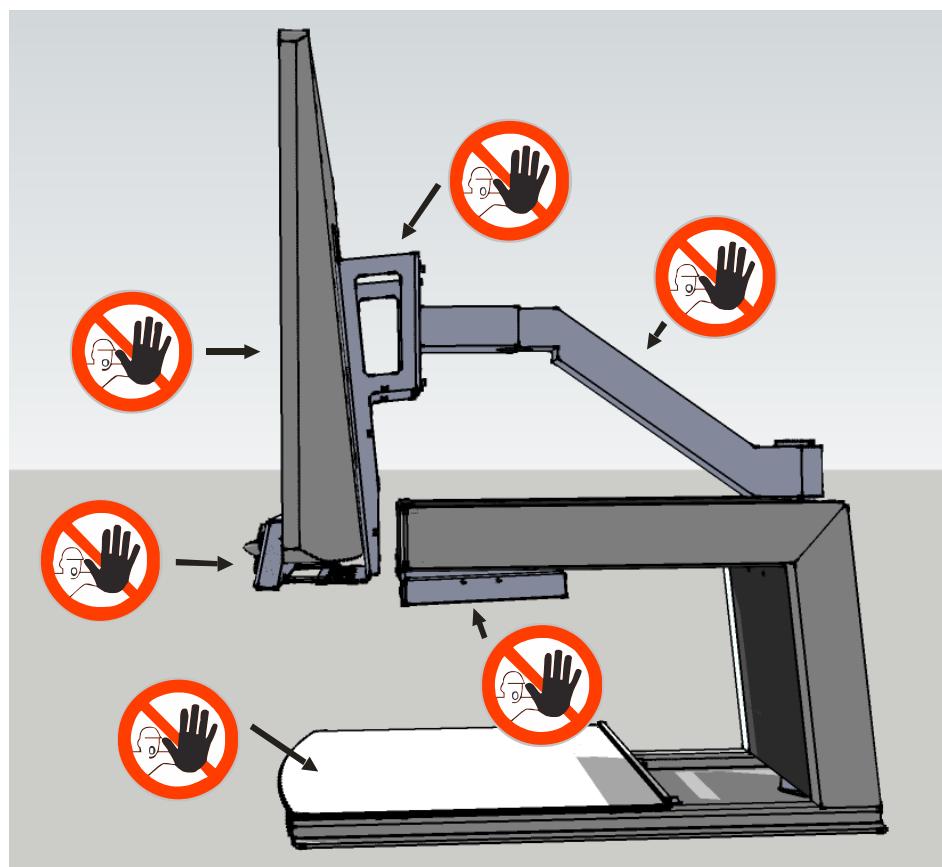

MAINTENANCE INSTRUCTIONS


- Occasionally clean the screen and the lens using a damp cloth. For further instructions: check the screen manual.
- Treat your device with care and you will enjoy it for years to come.

Chapter 2 : Installation

The Vocatex is solely to be installed by authorised personnel. You can find a list on www.kobavision.be

Chapter 3: Controls

OPERATING PANEL

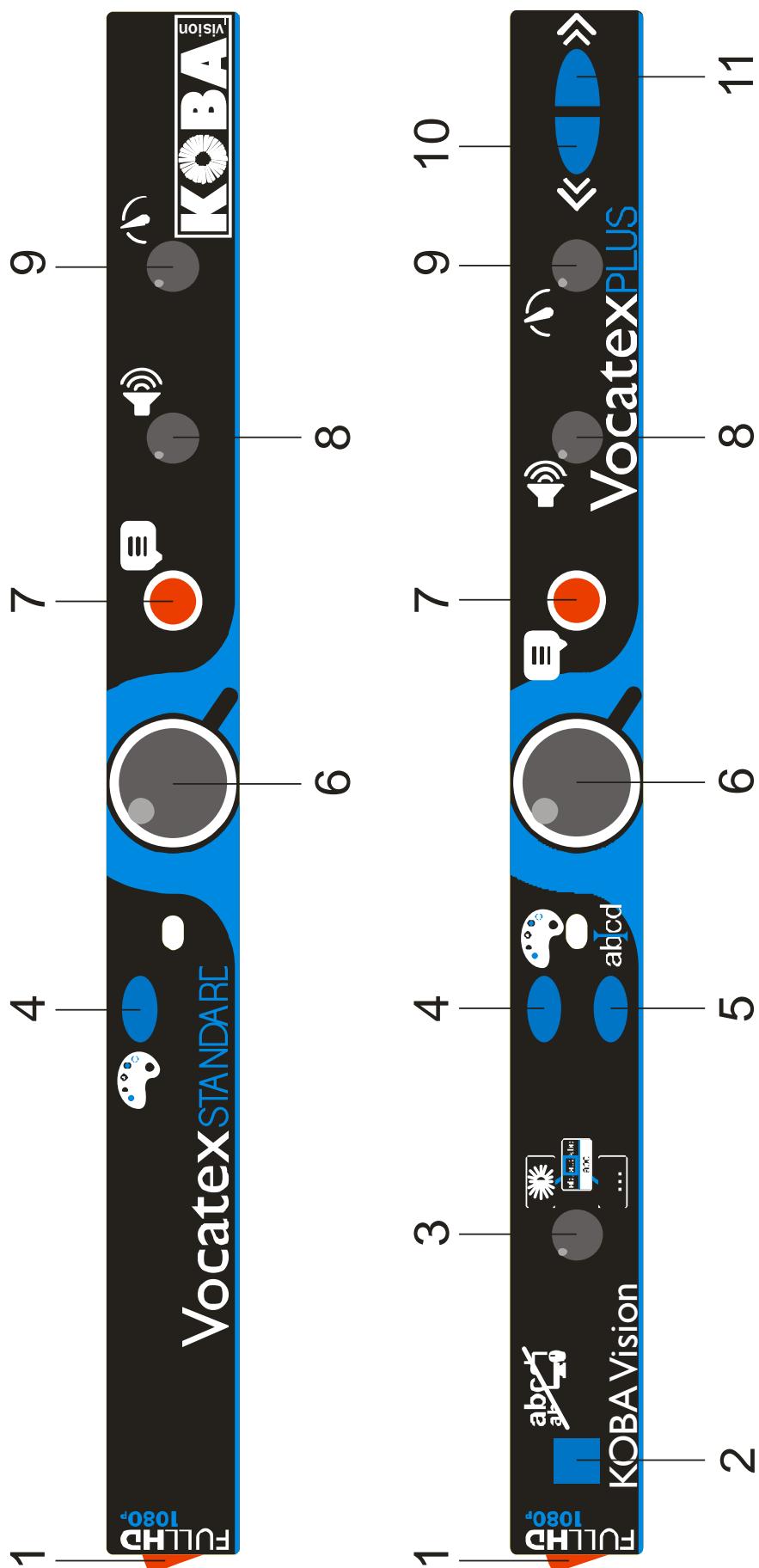1	ON/OFF switch
2	Switch computer/Vocatex
3	<p>Press: Switch between brightness/background, reading bar and ticker</p> <p>Turn in colour mode: brightness</p> <p>Turn in positive or negative reading : improve background</p>
4	<p>Press briefly: switch between photo mode, negative, positive reading</p> <p>(Vocatex Standard: the column selector is available in a 4th reading mode)</p> <p>Hold button: changes colour of the background or characters</p>
5	<p>Press briefly: hide/unhide column selector</p> <p>Hold button: change the colour of the column selector</p>
6	<p>Turn: zoom in/zoom out</p> <p>Press: overview/magnification</p> <p>Press long: lock auto focus</p>
7	Read aloud
8	<p>Turn: volume</p> <p>Press and turn: select another reading voice or automatic</p>
9	Reading speed
10	<p>Press briefly: go back one word in the text</p> <p>Hold button: go back one sentence in the text</p> <p>Press both navigation button 10 + 11: read full image aloud</p>
11	<p>Press briefly: go ahead one word in the text</p> <p>Hold button: go ahead one sentence in the text</p> <p>Press both navigation button 10 + 11: read full image aloud</p>

SWITCHING ON/OFF

The device is switched on with **button 1**. Most screens are switched on by the Vocatex. If this is not the case, switch on the screen as well (see screen manual).

The Vocatex is switched off with **button 1**. Some screens switch off the Vocatex when they are switched off.

After switching on, an on-screen bar shows the start-up progress. The Vocatex indicates when it is ready to be used by telling “the device is ready for use”.

READING ALOUD

Procedure

1. Put the text on the sliding table beneath the Vocatex.
2. The text now becomes visible on the screen.
3. Select a text block by placing the text block in the middle of the screen or by crossing the text block by the column selector.
4. Press **button 7** to start the read aloud function.

Usually I try to blot out other people's mobile phone conversations when I'm on a train, but there was something intriguing about this one. I was on an early Eurostar heading for Zurich via Paris, sitting across the aisle from a typical British banker, when he took a call from his wife. Even across the carriage I could hear her agitation at the other end of the line, and his attempts to hush the call only made it all the more interesting. He signed off by promising to call someone – which he then did.

New high-speed links have cut journey times significantly over the last two years, with the next big step forward due on 13 December, when new track brings Amsterdam an hour closer, as well as shortening trips to many other cities. Building on the next phase of Europe's high speed rail project begins next year.

And now that we have modern, high-speed services running to the heart of city destinations, trains are no longer the dour-earred refuge of enthusiasts. Thanks to

the corridor.

Although I'm not proud of the eavesdropping component, this tantalising glimpse of someone else's life is actually typical of travel by train. On a plane, you see nothing but clouds, and on a motorway you see the back end of the lorry in front. But a train allows insight into a destination, peering into a nation's backyards as it runs unfettered through country and city, transporting a cross-section of locals and travellers, some of whom may be disposed to talk – even if it's just on a mobile phone.

Eurostar itself is an unbeatable formula, especially with its starting point in the wonderful railway

1

2

1. Column Selector
2. Text block

Pause

Reading aloud is initiated by pressing **button 7**. To pause, press **button 7 or 8**. To resume reading, press **button 7 or 8**.

Pressing **button 7** will always initiate a new OCR process which will take longer. Pressing **button 8** will immediately start and pause the text without initiating a new OCR process. We recommend to use **button 8** while using the news ticker.

Continuous reading

The Vocatex reads continuously. When you move further into the text, the Vocatex will continue reading without interruption. The reading frame disappears when moving but will reappear when the new position is found.

Reading full image

The Vocatex can read a full image by pressing both navigation buttons (**button 10 and 11**) at the same time. The Vocatex will read the full image without column recognition. This features simplifies reading invoices, bank receipts and tables.

Navigation

The Vocatex Plus can navigate by word or by sentence. When pressing and holding **button 10** or **button 11** the device will go further or go back sentence by sentence. By pressing briefly you go further or go back word by word.

You can easily have the text read out again by sliding the text upwards again with the slide table. If you slide the text upwards by more than 1cm, the Vocatex will resume reading aloud from the start and not from the previous reading position.

Progress indicator

While reading, the Vocatex displays a progress indicator. This is a bar which is shown at the bottom of the screen and indicates the progress in the text. The progress indicator can appear in 3 different forms.

Reading within the same column

- Sliding up or continue reading without sliding the text that is being read out of screen: the reading frame disappears and reappears when the new position is found.

- Continue reading while the text is slid out at the top of the screen. The frame disappears and reappears when the text that is being read enters the screen again. When the text connects, this is shown by double bars that approach one and other.

- When it is impossible to reconnect, meaning that the text was slid through too far, an arrow appears to the right of the bar. You can position the text back until you see the indicator without the arrow.

Reading a next column

During the reading of one column you can already move to the next without interrupting reading of the previous column. In that case an arrow is shown next to the bar. To repeat a text you can slide the text downwards. The text will be read from the top.

Ticker

The ticker will display the readout text on the screen. The ticker can be activated or deactivated by pressing **button 3**. The size of the ticker is set by turning **button 3**. The position set by pressing and turning **button 3**.

The colours are configured automatically depending on the colour combinations that are set with **button 4**.

When changing the size of the ticker with **button 3**, the news ticker switches from scrolling to word by word.

The speed at which the text is displayed is linked to the reading speed of the Vociatex. You can adjust the speed by turning **button 9**.

Reading bar

The horizontal reading bar is a clear bar from left to right on the screen. The text that isn't covered by the bar is slightly visible. The reading bar is used to cover text or to select a part of the text to read aloud.

Press **button 3** until the bar is displayed on the screen. Turn **button 3** to set the position. Press and turn **button 3** to change the width. Press **button 5** to show or hide edge lines. Press long on **button 5** to change colours.

Press both navigation buttons **10 and 11** at the same time to read the text in the reading bar without column recognition.

Press **button 7** to read the text in the reading bar with column recognition.

Pitch control

Press the volume **button 8** and turn the speed button at the same time **button 9**.

Language and reading voices

Attention! This menu will pop-up when initializing the unit for the first time.

You can change the language options at any time. To open the language menu, you press **button 7** for 10 seconds. The language menu will open. The menu has 3 steps:

Step 1: Language of the spoken assisting messages

This is where you adjust the language of the spoken buttons. You choose your mother tongue. An inhabitant of France chooses French. By turning the **button 6** you choose the language. Hold **button 7** to go to step 2.

Step 2: Preferred voice

Most languages have several voices. In step 2 you choose your preferred voice for the spoken buttons and your preferred voice for reading out the texts by turning **button 6**. Usually you can choose between a male and female voice. In French, for example, you can choose between Virginie, Thomas, Felix and Audry. You select the language by turning the big knob. By holding **button 7** you will go to step 3.

Step 3: Other reading voices

If you would like to have texts read out by other voices or in different languages, you can select other voices in step 3. By turning **button 6** you select a voice. By briefly pressing **button 7** you can switch a voice on or off. (X) indicates that a voice is switched on. () indicates that a voice is switched off. You can switch on/off various voices. By holding **button 7** you will exit the voice and language menu.

Automatic language recognition

You can activate the option “automatic language recognition” by pressing and turning **button 8** until the Vocatex announces “automatic”.

When this function is set, each text bloc will be read in the according language. The default voice is predefined. If you want the text to be read in another voice, only select 1 voice per language (your preferred voice) in the language menu.

COMPUTER CONNECTION

Button 2 of the Vocatex switches between HDMI 1 and HDMI 2, or HDMI 1 and HDMI 3.

To use the button on the front panel, the Vocatex needs to be connected on HDMI 1, your computer needs to be connected on HDMI 2 or 3 of the screen. Connect with a HDMI-HDMI, HDMI-DVI or HDMI-DisplayPort cable.

The computer connection of the Vocatex has 2 possibilities:

Full screen (most common)

By pressing the pc button on the Vocatex, the screen switches to your computer. The full computer image is displayed on your screen. Recommended resolution: 1360x768 or 1280x720.

Split Screen

(in combination with ZoomText 10 and higher or SuperNova 15.03 and higher) – only Windows OS

With the split screen possibility you can use a part of the screen to display the Vocatex image and another part to display your computer. The video image can simply be put on top, bottom, left or right while having your computer image on the same screen. When working in split screen mode you will have full access to your computer.

The video image is sent with a USB second cable to your computer. Use the **button 2** to switch to full screen. Open ZoomText or SuperNova and start the camera function. The video image is displayed.

Specifications Computer

- Unused USB port (USB 3.0 or 2.0)
- Windows 10, 8.1, 8, 7 – 32/64 bit;
- ZoomText 10 and higher or SuperNova 15.03 and higher.

Installation

- Connect the Vocatex to the computer via the USB cable (USB 3.0 preferred);
- Configure the Vocatex as camera under ZoomText or SuperNova;
- Start ZoomText or SuperNova webcam.

PROGRAMMABLE BUTTONS

The Vocatex has programmable buttons. **Buttons 2, 3, 4 and 6** can have different functions and possibilities. These functions can be activated by your dealer.

Button 2:

- Switch between magnifier and computer (**default**)
- Reset

Button 3 (press shortly to navigate between the different possibilities)

- Brightness/background, Reading bar, Ticker (**default**)
- *Lighting, Reading line, Reading bar with ticker*

Button 4: Different colour combinations

- Black/White, White/Black, Black/Yellow, Yellow/Black, Black/Green, Green/Black (**default**)
- *Black/Red, Red/Black, Black/Blue, Blue/Black, Black/Cyan, Cyan/Black, Black/Magenta, Magenta/Black, Black/Orange, Orange/Black, Black/Gray, Gray/Black, Blue/Yellow, Yellow/Blue, Blue/White, White/Blue, Red/White, White/Red, Orange/White, White/Orange, Green/White, White/Green*

Button 6:

- Overview/magnification (**default**)
- *Switch between magnifier and computer, Reset, Reading aloud*

LIGHTING

(not set by default – ask your dealer to activate)

Press **button 3** until the Vocatex announces “lighting”.

By turning **button 3** you can set the brightness of the camera and button lights.

READING LINE

(not set by default – ask your dealer to activate)

The horizontal reading line is a line from left to right on the screen.

Press **button 3** until the line is displayed on the screen. Turn **button 3** to set the position. Press and turn **button 3** to change the colour.

READING BAR WITH TICKER

(not set by default – ask your dealer to activate)


Same as reading bar in combination with the ticker.

HEADPHONE

A headphone or external speakers can be connected at the Vocatex (see mini-jack connector on the back).

SLIDING TABLE


Under the table you can find a lever which controls the brake. If you move the lever to the left, the slide table is free to move left/right/up/down. If you move the lever to the right you adjust the friction. If you move the lever all the way to the right, the slide table will be blocked.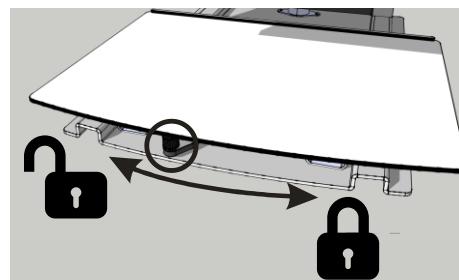

HEIGHT AND TILT ADJUSTMENT

Height (Flex arm and large screens)

According to personal preference, the screen of the Vocatech can be moved up or down. To do this, make sure you take hold of the screen on both the left and right side of the screen.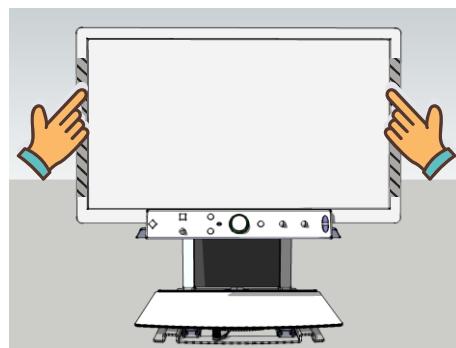


Increase Lift Strength (only flex arm)

If the screen doesn't stay up when raised, increase the lift strength by turning the angle key right.

Decrease Lift Strength (only flex arm)

If the screen doesn't stay down when lowered, decrease the lift strength by turning the angle key left.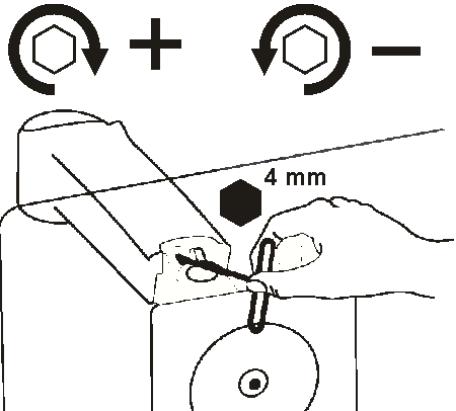

Tilt (Flex arm)


According to personal preference, the screen of the Vocatech can be tilted. To do this, make sure you take hold of the screen on both the left and right side of the screen.

Increase Friction

If the screen tilts down, then you'll need to increase friction by turning the angle key right.

Decrease Friction

If the screen is too difficult to tilt, then you'll need to decrease friction by turning the angle key left.

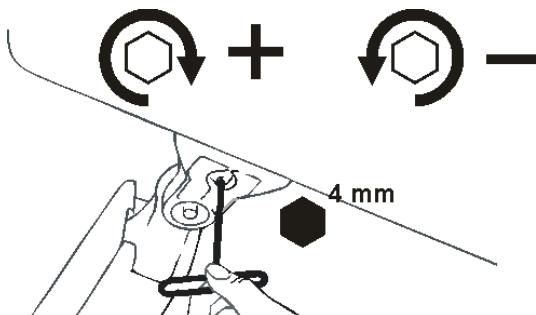USAGE TIPS

Reading aloud – Text blocks

Reading aloud is best done in a reading mode where you easily can select the text blocks.

Reading aloud – Recognition

Text recognition isn't 100% accurate. The devices will make mistakes. By changing the magnification you can adapt the quality of recognition.

Reading

Reading is best done in positive or negative mode. These modes adapt the image for the visually impaired to a high-contrast image with improved background. Background and letters can be set at your discretion. The background can be set by means of background suppression (Vocatex Plus). This will allow you to filter out the structure of a sheet. This unique KOBA Vision function allows you at all times to set a high contrast background without annoying paper impurities. A newspaper can be read without annoying paper impurities. Factory settings will take you back to the original settings.

Watch photos

Photos are best watched in photo mode. This mode is a unprocessed image without contrast enhancement. By changing the brightness (Vocatex Plus), you can adjust the image so that the photo becomes clearly visible. Quality is slightly improved when locking the auto focus.

Crafting

Crafting is best done with a small magnification. It is better to select a dark background during this activity so that the camera is not hindered by large differences in contrast with whatever is darkened by the hands.

Writing

You can also write under your video magnifier. It is best to use a black pen and lined paper and to lock the auto focus on the paper.

Chapter 4: Troubleshooting

Below is a list of problems that may occur with your Vocatex. Consult the list first before contacting KOBA Vision or your dealer.

Symptom	Symptom	Symptom
The device isn't turning on	No display on the screen	No sound
Possible causes	Possible causes	Possible causes
No power	Power failure The screen is not switched on	The screen volume is on zero The sound has been disrupted The headphone is plugged in The HDMI connector is loose
What to do	What to do	What to do
Check the power cable Check the adapter and verify if a green led is burning	Check whether the plug of the power cord has been properly plugged into the wall socket If the device still won't work, pull the plug out of the wall socket. Then wait 60 seconds before plugging it back into the wall socket. Turn the screen on again. Press the "on" button on the remote	Slightly turn up the screen volume Check whether the sound has not been disrupted. Point the remote control directly towards the sensor at the front of the screen. Unplug the headphone Check the HDMI plug.

Chapter 5: Warranty

WARRANTY

KOBA Vision offers a two-year warranty on every KOBA VISION video magnifier, starting from the delivery date. To be able to benefit from the warranty, you need to keep the box and the packaging material.

Warranty services are provided by KOBA Vision or dealers acting on behalf of KOBA Vision. When the video magnifier cannot be repaired correctly, KOBA Vision reserves the right to replace the reading device by an equal device. All components and products replaced within the warranty period become property of KOBA Vision.

WARRANTY PROVISIONS

The warranty becomes invalid if the video magnifier:

- Was not used according to instructions or was misused;
- Was damaged as a result of a fall or impact;
- Was disassembled or tinkered with;
- Was repaired earlier by a non-official service partner.

Consequential loss is under no circumstances covered by the warranty.

KOBA Vision will neither assume any liability whatsoever nor accept any possibility of liability except for the warranty provisions explicitly stated above. This limitation to the liability does not imply a breach of the customer's legal consumer rights or his or her rights vis-à-vis the vendor.

SERVICE FULFILMENT

Before claiming the warranty or service and before contacting KOBA Vision or a dealer, you must note product name, serial number, purchase date, invoice number and any error messages or particular problem aspects.

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